

Topi-CLICK® 35

Improving Life Through Innovative Pharmacy Solutions

35 mL Specifications

- The Topi-CLICK is a registered FDA device, Class I Dispenser, Product Code “KYX”. The Topi-CLICK is made of all FDA approved materials. The materials used are in compliance with FDA 21.177 code, which is the requirement for United States Pharmacopeia (USP). Topi-CLICK’s FDA Registration # is 3007142798 and Topi-CLICK’s FDA Device Listing Number is D003073.
- Each click (¼ turn of the Topi-CLICK® container base) moves the plunger (topical elevator) approximately 0.25ml of volume space, thereby dispensing 0.25ml of cream on average. (The dispensing mechanism is similar to a syringe, but with more control of the plunger movement, so Topi-CLICK® is easier to use.)
- The most common number of clicks used for one dose is 2 clicks (moves the plunger 0.5ml volume space); so when filling with approximately 35ml the Topi-CLICK® will provide approximately 64 doses (at 2 clicks per dose or 128 clicks total); the rest (approx. 0.75ml) is used for initial priming and (approx. 2.5ml) is residual.*
- The Topi-CLICK® container can be used to dispense a total amount of approximately 5ml to 35ml of a topical preparation depending on the volume to weight ratio of base and formula used.
- The Topi-CLICK® labels like a prescription vial – simplified labeling for lower cost – and easier to read.
- Domed applicator pad for application without dispensing on to hands first.
- Audible, visual and tactile click registration for user recognition.
- A “Refill” reminder system helps patients to reorder in a timelier manner. See Operating Instructions above.
- Designed to resist accidental discharge in purse, pocket or luggage.
- Made of FDA grade plastic. UV block in plastic protects the contents from UV rays.
- One time use only. Note: check with your State Board of Pharmacy to see if you are allowed to recycle prescription drug containers. CLICK here for more information on recycling Topi-CLICK®. Recycle Code is #5



***As required for compounded prescription pharmaceuticals dispensed in volume dispensers; verify the weight to volume conversions thoroughly.**



Order at www.doselogix.com or Call 877.870.8448

