

## Topi-CLICK<sup>®</sup> 35 Specifications



### Topi-CLICK<sup>®</sup> 35: 35 mL, 0.25 mL per CLICK

**DESCRIPTION:** Topi-CLICK<sup>®</sup> 35 is a smart, effective topical dosing applicator designed to overcome the various limitations associated with pumps, syringes, tubes, tubs and jars. The Topi-CLICK<sup>®</sup> 35 can be used to dispense approximately 5 mL to 35 mL of a topical preparation depending on the volume to weight ratio of base and formula used.

Topi-CLICK<sup>®</sup> makes it easy to accurately measure the dose and apply topical medications to the intended area, thereby decreasing the risk of improper dosing and contamination while improving compliance and outcomes of therapy.

### FEATURES:

- Each CLICK or turn of the base moves the plunger approximately 0.25 mL of volume space, thereby dispensing 0.25 mL of cream on average. The most common number of CLICKs used for one dose is 2 CLICKs or 0.50 mL of volume space.
- Audible, visual and tactile CLICK registrations for user recognition.
- Lower cost with simplified labeling – labels as fast and easily as a prescription bottle.
- Built-in applicator to prevent medication transference to unintended areas.
- “Refill” reminder badge helps patients to reorder in a timely manner.
- Designed to resist accidental discharge in purse, pocket or luggage.

### TECHNICAL SPECIFICATIONS: Topi-CLICK<sup>®</sup> 35

<b>Dispenser Volume:</b>	Up to 37 mL
<b>Delivery Volume:</b>	Up to 34 mL
<b>Dispense per CLICK:</b>	0.25 mL
<b>Approximate Residual:</b>	2.5 mL
<b>Approximate Priming:</b>	0.50 mL or 2 CLICKs
<b>Doses Provided:</b>	When filling with 35 mL, the Topi-CLICK <sup>®</sup> will provide approximately 32 mL or 64 doses (at 2 CLICKs per dose or 128 CLICKs total).

- Registered FDA device, Class 1 Dispenser.
- Topi-CLICK's FDA Registration # is 3007142798 and the Device Listing Number is D003073.
- Materials compliant with FDA 21.177 code, which is the requirement for United States Pharmacopeia (USP <661>).
- Made of FDA grade materials | BPA and Phthalate free | UV block.
- Recycling Code is #5 ♻️. Visit <http://doselogix.com/recycling-your-topi-click> for more information on recycling Topi-CLICK<sup>®</sup>.

**\*Volumes are approximate. As required for compounded prescription pharmaceuticals dispensed in volume dispensers, verify the weight to volume conversions thoroughly.** See dispensing instructions at <http://doselogix.com/downloads-how-it-works>. One time use only.

**Topi-CLICK<sup>®</sup> 35 is the most accurate topical dosing dispenser among those compared in an independent study titled *Topical Metered Dosing Dispenser Performance Evaluation by Analytical Research Laboratories*. Request study at [http://doselogix.com/accuracy\\_study/](http://doselogix.com/accuracy_study/).**

## Topi-CLICK® Topi-CLICK® 35 3Port™ Specifications



**Topi-CLICK® 35 3Port™: 35 mL, 0.25 mL per CLICK**

**DESCRIPTION:** The Topi-CLICK® 35, 3Port™ uses the same technology as the Topi-CLICK® 35 – but with extra ports for faster and easier CLICK-Dosing™ of thicker medications. The advantages for the patient are added convenience and ease-of-use when dealing with the thickest or refrigerated topical preparations.

The Topi-CLICK® 35 3Port™ can be used to dispense approximately 5 mL to 35 mL of a topical preparation depending on the volume to weight ratio of base and formula used.

*Designed and recommended for use with ONLY the thickest formulations.*

### FEATURES:

- Each CLICK or turn of the base moves the plunger approximately 0.25 mL of volume space, thereby dispensing 0.25 mL of cream on average. The most common number of CLICKs used for one dose is 2 CLICKs or 0.50 mL volume space.
- Audible, visual and tactile CLICK registrations for user recognition.
- Lower cost with simplified labeling – labels as fast and easily as a prescription bottle.
- Built-in applicator to prevent medication transference to unintended areas.
- “Refill” reminder badge helps patients to reorder in a timely manner.
- Designed to resist accidental discharge in purse, pocket or luggage.

### TECHNICAL SPECIFICATIONS: Topi-CLICK® 35 3Port™


<b>Dispenser Volume:</b>	Up to 37 mL
<b>Delivery Volume:</b>	Up to 34 mL
<b>Dispense per CLICK:</b>	0.25 mL
<b>Approximate Residual:</b>	2.5 mL
<b>Approximate Priming:</b>	0.50 mL or 2 CLICKs
<b>Doses Provided:</b>	When filling with 35 mL, the Topi-CLICK® will provide approximately 32 mL or 64 doses (at 2 CLICKs per dose or 128 CLICKs total.)

- Registered FDA device, Class 1 Dispenser.
- Topi-CLICK’s FDA Registration # is 3007142798 and the Device Listing Number is D003073.
- Materials compliant with FDA 21.177 code, which is the requirement for United States Pharmacopeia (<USP 661>).
- Made of FDA grade materials | BPA and Phthalate free | UV block.
- Recycling Code is #5 ♻️. Visit <http://doselogix.com/recycling-your-topi-click> for more information on recycling Topi-CLICK®.

**\*Volumes are approximate. As required for compounded prescription pharmaceuticals dispensed in volume dispensers, verify the weight to volume conversions thoroughly.** See dispensing instructions at <http://doselogix.com/downloads-how-it-works>. One time use only.

**Topi-CLICK® 35 is the most accurate topical dosing dispenser among those compared in an independent study titled *Topical Metered Dosing Dispenser Performance Evaluation by Analytical Research Laboratories*. Request study at [http://doselogix.com/accuracy\\_study/](http://doselogix.com/accuracy_study/).**

## Topi-CLICK® Topi-CLICK® 35 DOSE ™ Specifications

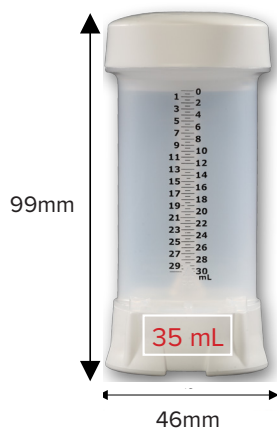

### Topi-CLICK® 35 DoseCheck™: 35 mL, 0.25 mL per CLICK

**DESCRIPTION:** Healthcare providers and caregivers can easily verify a dose with the NEW graduated Topi-CLICK® 35 dose check system.

The Topi-CLICK® 35 DOSE ™ topical dosing dispenser provides the same advanced technology as the Topi-CLICK® 35 and offers EXTRA assurance with graduated marks so the dose starting and stopping points can be easily verified and recorded.

Topi-CLICK makes it easy to accurately measure the dose and apply topical medications to the intended area, thereby decreasing the risk of improper dosing and contamination while improving compliance and outcomes of therapy.

**Topi-CLICK® 35 DoseCheck™ initial start point should begin when the bottom of the plunger is at the 30 mL graduated mark.** The DoseCheck™ can hold the same amount as a Topi-CLICK® 35 but is specialized for use in settings where dose tracking is required.

#### FEATURES:

- Graduated marks at each 0.50 mL.
- Each CLICK or turn of the base moves the plunger approximately 0.25 mL of volume space, thereby dispensing 0.25 mL of cream on average. The most common number of CLICKs used for one dose is 2 CLICKs or 0.50 mL volume space.
- Filling: Assuming two priming CLICKs— when filling the Topi-CLICK® DoseCheck™, it is recommended to add an additional 4 mL per prescription to allow for priming, data recording marks and residual. There will normally be an additional one or two CLICKs of dispense (0.25 to 0.50 mL) after the plunger advances to the “0” mark.
- Audible, visual and tactile CLICK registrations for user recognition.
- Lower cost with simplified labeling – labels as fast and easily as a prescription bottle.
- Built-in applicator to prevent medication transference to unintended areas.
- “Refill” reminder badge helps patients to reorder in a timely manner.
- Designed to resist accidental discharge in purse, pocket or luggage.

#### TECHNICAL SPECIFICATIONS: Topi-CLICK® 35 DOSECHECK™

<b>Dispenser Volume:</b>	Up to 34 mL
<b>Delivery Volume:</b>	30 mL (based on beginning initial start point at the 30 mL graduated mark)
<b>Dispense per CLICK:</b>	0.25 mL
<b>Approximate Residual:</b>	3.50 mL (Residual remaining after plunger reaches the “0” graduated mark.)
<b>Approximate Priming:</b>	0.50 mL or 2 CLICKs
<b>Doses Provided:</b>	The Topi-CLICK DoseCheck will provide approximately 30 mL or 60 doses (at 2 CLICKs per dose or 120 CLICKs total) between the “30” and “0” graduated marks. 1 mL of cream can still be dispensed from the DoseCheck after reaching “0”. The 1 mL buffer space is there to ensure accurate visual dose tracking and dispensation is maintained for the patient.

- Registered FDA device, Class 1 Dispenser.
- Topi-CLICK’s FDA Registration # is 3007142798 and the Device Listing Number is D309579.
- Materials compliant with FDA 21.177 code, which is the requirement for United States Pharmacopeia (USP <661>).
- Made of FDA grade materials | BPA and Phthalate free | UV block.
- Recycling Code is #5 . Visit <http://doselogix.com/recycling-your-topi-click> for more information on recycling Topi-CLICK®.

**\*Volumes are approximate. As required for compounded prescription pharmaceuticals dispensed in volume dispensers, verify the weight to volume conversions thoroughly.** See dispensing instructions at <http://doselogix.com/downloads-how-it-works>. One time use only.

**Topi-CLICK® 35 is the most accurate topical dosing dispenser among those compared in an independent study titled *Topical Metered Dosing Dispenser Performance Evaluation by Analytical Research Laboratories*. Request study at [http://doselogix.com/accuracy\\_study/](http://doselogix.com/accuracy_study/).**

## Topi-CLICK® Topi-CLICK® 35 USP671™ Specifications



### Topi-CLICK® 35 USP671™: 35 mL, 0.25 mL per CLICK

**DESCRIPTION:** Topi-CLICK® 35 USP671™ is USP Compliant to USP671 (250nm to 450 nm) and blocks UVA, UVB and Violet Light.

Topi-CLICK® 35 USP671™ is a smart, effective topical dosing applicator designed to overcome the various limitations associated with pumps, syringes, tubes, tubs and jars. The Topi-CLICK® 35 USP671™ can be used to dispense approximately 5 mL to 35 mL of a topical preparation depending on the volume to weight ratio of base and formula used.

Topi-CLICK® makes it easy to accurately measure the dose and apply topical medications to the intended area, thereby decreasing the risk of improper dosing and contamination while improving compliance and outcomes of therapy.

#### FEATURES:

- Each CLICK or turn of the base moves the plunger approximately 0.25 mL of volume space, thereby dispensing 0.25 mL of cream on average. The most common number of CLICKs used for one dose is 2 CLICKs or 0.50 mL volume space.
- Audible, visual and tactile CLICK registrations for user recognition.
- Lower cost with simplified labeling – labels as fast and easily as a prescription bottle.
- Built-in applicator to prevent medication transference to unintended areas.
- “Refill” reminder badge helps patients to reorder in a timely manner.
- Designed to resist accidental discharge in purse, pocket or luggage.

#### TECHNICAL SPECIFICATIONS: Topi-CLICK® 35 USP671™

<b>Dispenser Volume:</b>	Up to 37 mL
<b>Delivery Volume:</b>	Up to 34 mL
<b>Dispense per CLICK:</b>	0.25 mL
<b>Approximate Residual:</b>	2.5 mL
<b>Approximate Priming:</b>	0.50 mL or 2 CLICKs
<b>Doses Provided:</b>	When filling with 35 mL, the Topi-CLICK® will provide approximately 32 mL or 64 doses (at 2 CLICKs per dose or 128 CLICKs total).

- Registered FDA device, Class 1 Dispenser.
- Topi-CLICK’s FDA Registration # is 3007142798 and the Device Listing Number is D003073.
- Materials compliant with FDA 21.177 code, which is the requirement for United States Pharmacopeia (USP <661>).
- Made of FDA grade materials | BPA and Phthalate free.
- Topi-CLICK® 35 USP671™ is USP Compliant to USP671 (250nm to 450 nm) and blocks UVA, UVB and Violet Light.
- Recycling Code is #5 ♻️. Visit <http://doselogix.com/recycling-your-topi-click> for more information on recycling Topi-CLICK®.

**\*Volumes are approximate. As required for compounded prescription pharmaceuticals dispensed in volume dispensers, verify the weight to volume conversions thoroughly.** See dispensing instructions at <http://doselogix.com/downloads-how-it-works>. One time use only.

**Topi-CLICK® 35 is the most accurate topical dosing dispenser among those compared in an independent study titled *Topical Metered Dosing Dispenser Performance Evaluation by Analytical Research Laboratories*. Request study at [http://doselogix.com/accuracy\\_study/](http://doselogix.com/accuracy_study/).**

## Topi-CLICK® Perl™ Vaginal Dosing Kit



### Topi-CLICK® 35 Perl Vaginal Dosing Kit: 35 mL, 0.25 mL per CLICK

**DESCRIPTION:** Topi-CLICK Perl™ (Precise / Easy / Reliable / Loading) is an innovative new way to dispense vaginal medications. The result of combining the most accurate dispenser revealed in an independent study<sup>1</sup>, the Topi-CLICK® 35, with the new patent pending Perl™ Snap-On Applicator, is a precise vaginal dosing method designed for comfort.

The Topi-CLICK® 35 Perl™ can be used to dispense approximately 5 mL to 35 mL of a topical preparation depending on the volume to weight ratio of base and formula used.

#### FEATURES:

- Each CLICK or turn of the base moves the plunger approximately 0.25 mL of volume space, thereby dispensing 0.25 mL of cream on average. The most common number of CLICKs used for one dose is 2 CLICKs or 0.50 mL volume space.
- Audible, visual and tactile CLICK registrations for user recognition.
- Lower cost with simplified labeling – labels as fast and easily as a prescription bottle.
- Built-in applicator to prevent medication transference to unintended areas.
- “Refill” reminder badge helps patients to reorder in a timely manner.
- Designed to resist accidental discharge in purse, pocket or luggage.

#### TECHNICAL SPECIFICATIONS: Topi-CLICK® 35 Perl Vaginal Dosing Kit

<b>Dispenser Volume:</b>	Up to 37 mL
<b>Delivery Volume:</b>	Up to 34 mL
<b>Dispense per CLICK:</b>	0.25 mL
<b>Approximate Residual:</b>	2.5 mL
<b>Approximate Priming:</b>	0.50 mL or 2 CLICKs
<b>Doses Provided:</b>	When filling with 35 mL, the Topi-CLICK® will provide approximately 32 mL or 64 doses (at 2 CLICKs per dose or 128 CLICKs total.)


- Registered FDA device, Class 1 Dispenser.
- Topi-CLICK’s FDA Registration # is 3007142798 and the Device Listing Number is D003073.
- Materials compliant with FDA 21.177 code, which is the requirement for United States Pharmacopeia (USP <661>).
- Made of FDA grade materials | BPA and Phthalate free | UV block.
- Recycling Code is #5 . Visit <http://doselogix.com/recycling-your-topi-click> for more information on recycling Topi-CLICK®.

**\*Volumes are approximate. As required for compounded prescription pharmaceuticals dispensed in volume dispensers, verify the weight to volume conversions thoroughly.** See dispensing instructions at <http://doselogix.com/downloads-how-it-works>. One time use only.

**Topi-CLICK® 35 is the most accurate topical dosing dispenser among those compared in an independent study titled *Topical Metered Dosing Dispenser Performance Evaluation by Analytical Research Laboratories*. Request study at [http://doselogix.com/accuracy\\_study/](http://doselogix.com/accuracy_study/).**



## Topi-CLICK® 35, Perl Vaginal Snap-on Applicator™ Specifications

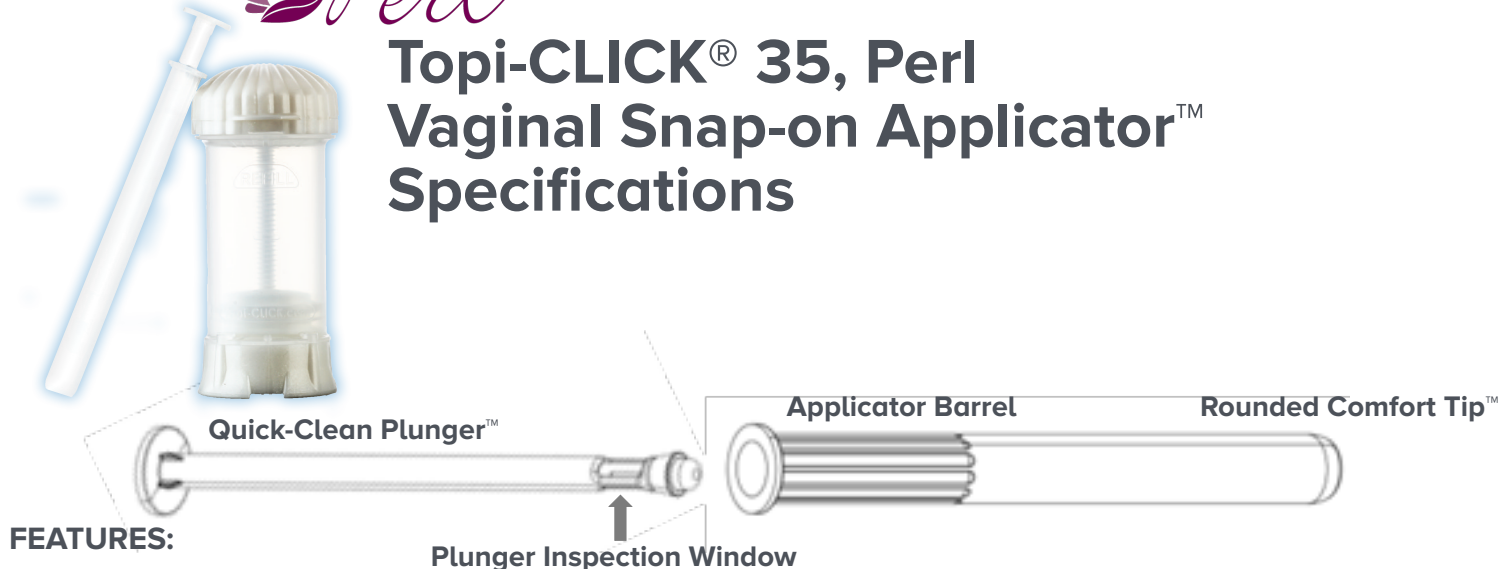



### Topi-CLICK® 35, Perl™ Snap-on Applicator Features



- Easy & Accurate Dosing™ compared to older style of threaded tip applicators for filling by tubes.
- Rounded Comfort Tip™ to help insertion be more pain-free and safer for patients.
- Snap-On Applicator Tip™ for fastest connecting and disconnecting during loading.
- Improved hygiene: Elimination of screw threads at tip allow for more thorough and quick cleaning.
- Quick-Clean Plunger™ design for easier cleaning.
- Maximum Applicator Capacity = 4.75 mL
- Minimized Residual as compared to older style of threaded tip applicators for filling by tubes.

Filled Amount	Threaded Ribbed	Threaded Smooth	Perl (Non-Threaded)
Filled to 1.0 mL: Residual =	0.12 mL	0.20 mL	0.05 mL
Filled to 4.0 mL: Residual =	0.19 mL	0.22 mL	0.11 mL

#### ◆Applicator Barrel Material:

- Meets USP Class VI & ISO-10993
- BPA, PVC, PCB, and Phthalate Free 
- Reusable and does not degrade after 30 dishwasher cycles.
- Recycle code 4  LDPE

#### ◆Quick-Clean Plunger™ Material:

- Meets USP Class VI & ISO-10993
- BPA, PVC, PCB, and Phthalate Free 
- Reusable and does not degrade after 30 dishwasher cycles.
- Recycle code 5  PP

\*Volumes are approximate. As required for compounded prescription pharmaceuticals dispensed in volume dispensers; verify the weight to volume conversions thoroughly.

Topi-CLICK® 35 is the most accurate topical dosing dispenser among those compared in an independent study titled *Topical Metered Dosing Dispenser Performance Evaluation* by Analytical Research Laboratories. Request study at [http://doselogix.com/accuracy\\_study/](http://doselogix.com/accuracy_study/).

## Topi-CLICK® MICRO™

## Topi-CLICK® Micro™ Specifications



**Topi-CLICK® Micro™: 9 mL, 0.05 mL per CLICK**

**DESCRIPTION:** Topi-CLICK® Micro™ is a revolutionary way to accurately and easily micro-dose topical medications for humans and pets. The innovative triad-shaped dispenser allows for precision handling during application. Audible, Tactile and Visual CLICKs indicate measure of dose of .05 mL at each CLICK point of the unique triad shaped base.

The selection of 3 applicator styles provides customized dosing for patients.

Select one of 3 applicators



*Rounded Applicator™*



*Soft Angled Applicator™*



*Pin Point Applicator™*

Topi-CLICK® Micro™ Quick-fill Technology makes filling quick and simple for compounders.

### FEATURES:

- Each CLICK or turn of the triad-shaped container base moves the plunger approximately 0.05 mL of volume space, thereby dispensing 0.05 mL of cream on average. The most common number of CLICKs used for one dose is 2 CLICKs or 0.10 mL volume space.
- Audible, visual and tactile CLICK registrations for user recognition.
- Lower cost with simplified labeling – labels as fast and easily as a prescription bottle.
- Built-in applicator to prevent medication transference to unintended areas.
- “Refill” reminder badge helps patients to reorder in a timely manner.
- Designed to resist accidental discharge in purse, pocket or luggage.

### TECHNICAL SPECIFICATIONS: Topi-CLICK® Micro™

<b>Dispenser Volume:</b>	Up to 9.75 mL
<b>Delivery Volume:</b>	Up to 9.45 mL
<b>Dispense per CLICK:</b>	0.05 mL
<b>Approximate Residual:</b>	0.15 mL
<b>Approximate Priming:</b>	0.15 mL or 3 CLICKs
<b>Doses Provided:</b>	When filling with 9.3 mL the Topi-CLICK® will provide approximately 90 doses (at 2 CLICKs per dose or 180 CLICKs total).

- Registered FDA device, Class 1 Dispenser.
- Topi-CLICK’s FDA Registration # is 3007142798 and the Device Listing Number is D003073.
- Materials compliant with FDA 21.177 code, which is the requirement for United States Pharmacopeia (<USP 661>).
- Made of FDA grade materials | BPA and Phthalate free | UV block.
- Recycling Code is #7 ♻️. Visit <http://doselogix.com/recycling-your-topi-click> for more information on recycling Topi-CLICK®.

**\*Volumes are approximate. As required for compounded prescription pharmaceuticals dispensed in volume dispensers; verify the weight to volume conversions thoroughly.** See dispensing instructions at <http://doselogix.com/downloads-how-it-works>. One time use only.

## Topi-CLICK® Topi-CLICK® 140 Specifications



### Topi-CLICK® 140: 140 mL, 0.50 mL per CLICK

**DESCRIPTION:** The NEW Topi-CLICK® 140 is an intelligently designed topical dosing device that overcomes the various limitations associated with pumps, syringes, tubes, tubs, and jars. Topi-CLICK brand metered dosing applicators make it easy to accurately measure the dose and apply topical formulations to the intended area, thereby decreasing the risk of improper dosing and contamination while improving compliance and outcomes of therapy.

The Topi-CLICK® 140 offers the same great dosing benefits as the Topi-CLICK® 35 but is designed for larger-volume topical formulations up to 140 mLs. The friction-free 4Port™ domed applicator design allows dispensing of larger doses with ease and convenience.

### FEATURES:

- Each CLICK or turn of the base moves the plunger approximately 0.50 mL of volume space, thereby dispensing 0.50 mL of cream on average. The most common number of CLICKs used for one dose is 2 CLICKs or 1.0 mL volume space.
- Audible, visual and tactile CLICK registrations for user recognition.
- Lower cost with simplified labeling – labels as fast and easily as a prescription bottle.
- Built-in applicator to prevent medication transference to unintended areas.
- “Refill” reminder badge helps patients to reorder in a timely manner.
- Designed to resist accidental discharge in purse, pocket or luggage.

### TECHNICAL SPECIFICATIONS: Topi-CLICK® 140

<b>Dispenser Volume:</b>	Up to 140 mL
<b>Delivery Volume:</b>	Up to 135 mL
<b>Dispense per CLICK:</b>	0.50 mL
<b>Approximate Residual:</b>	4.0 mL
<b>Approximate Priming:</b>	1.0 mL or 2 CLICKs
<b>Doses Provided:</b>	When filling with 140 mL the Topi-CLICK® will provide approximately 135 doses (at 2 CLICKs per dose or 270 CLICKs total.).

- Registered FDA device, Class 1 Dispenser.
- Topi-CLICK’s FDA Registration # is 3007142798 and the Device Listing Number is D003073.
- Materials compliant with FDA 21.177 code, which is the requirement for United States Pharmacopeia (<USP 661>).
- Made of FDA grade materials | BPA and Phthalate free | UV block.
- Recycling Code is #5 ♻️. Visit <http://doselogix.com/recycling-your-topi-click> for more information on recycling Topi-CLICK®.

**\*Volumes are approximate. As required for compounded prescription pharmaceuticals dispensed in volume dispensers; verify the weight to volume conversions thoroughly.** See dispensing instructions at <http://doselogix.com/downloads-how-it-works>. One time use only.