

Filling Instructions*

Read all information, specifications and instructions carefully before filling and dispensing in Topi-CLICKs.

Step 1



Fill Topi-CLICK with desired amount using Topi-CLICK Filling Station, or EMP jar & nozzle, or spatula.

Step 1a



If necessary, stop every third of the fill to tap the Topi-CLICK on the palm of your hand or counter to help settle the topical.

Step 2



Snap applicator pad (cap with hole) on Topi-CLICK tube by aligning inside key tabs with slots on top-edge of tube and snap tight.

Step 3



Prime the Topi-CLICK by turning the bottom until topical is dispensed and wipe off topical.

Step 3a: LEAVE IN HOME / START POSITION



Priming: When rotating base, watch the Visual Indicators

Indicator Node on body is fixed at the CLICK point



Indicator Column moves with Base. This shows Node in-line with column.

The HOME / START POSITION

VERY IMPORTANT TO LEAVE IN HOME POSITION WHICH IS THE START POSITION OF NEXT DOSE

Step 4



Snap on cover cap. (if cover cap doesn't fit tightly, it indicates applicator pad in Step 2 is not snapped all the way down).

Step 5



Apply center of label to side of the Topi-CLICK container opposite of the "REFILL" and graduation marks.

For more information on dispensing amounts per click see the [Specifications](#) document or see www.Topi-CLICK.com. See the [User Instructions](#) sheet on how to apply with Topi-CLICKs.

Notes: ¹Each CLICK (¼ turn of the Topi-CLICK base) moves the plunger up approximately 0.25ml inside tube.

²Allow for extra topical needed for priming (0.75mL) and residual (2.5mL).

³The graduation marks are for reference only & can be used with the bottom of the plunger.

⁴Some bases are not practical to use with the Topi-CLICK.

*As required for compounded prescription pharmaceuticals; verify formulas and weight per volume.

For more information and to download this page see www.Topi-CLICK.com.

