

## How to Use the Topi-CLICK<sup>®</sup> Perl<sup>™</sup> Vaginal Dosing Kit

### 1. Opening / Attaching the Snap-on Applicator<sup>™</sup> to the Perl

1. Flip Perl shell cap open using integrated tab. (Figure a.)
2. Snap the Snap-on Applicator to the extended receptor port on the Perl shell. (Figure b.) **Note:** The Snap-on Applicator Plunger should be fully inside the applicator barrel when dosing as shown. (Figure b and c.)

### 2. Loading Perl / Dispensing Medication

1. To dispense medication, simply turn/CLICK the base of the Perl to load the metered dose. **Note:** For thicker topicals, allow 15 to 30 seconds for full dose to load into the Snap-on Applicator. (Figure c.)

**\* It is important to keep the Perl in vertical position when loading the Snap-on Applicator<sup>™</sup> as shown. (Figure c.)**

#### Dosing Notes:

Each turn/CLICK dispenses 0.25 mL of medication accurately and does not need fill lines on the Snap-on Applicator. (see table below)

Dosing Guide
1 CLICK = 0.25 mL
2 CLICKs = 0.50 mL
3 CLICKs = 0.75 mL
4 CLICKs = 1.00 mL

#### No Reference Lines on Applicator:

Topi-CLICK is the most accurate topical dispenser among those tested in an independent study\*. Unlike the traditional tube and applicator, the Snap-on Applicator was designed to be loaded by Topi-CLICK technology and thereby does not need reference lines to ensure accuracy.


\* Go to [www.DoseLogix.com/accuracy\\_study](http://www.DoseLogix.com/accuracy_study) to request a free copy of the independent accuracy study.

### 3. Removing the Snap-on Applicator


1. Simply tilt the Snap-on Applicator over as shown in (Figure d.) until the applicator releases from the receptor extended port on the shell.
2. **Reminder:** Depending on the patients removal technique, there may be a small dollop of cream remaining on the Snap-on Port which is part of the patient's dose.




- ◆ USP Class VI compliant
- ◆ BPA/Latex Free
- ◆ Applicator is reusable and performance does not degrade after 30 dishwasher cycles.

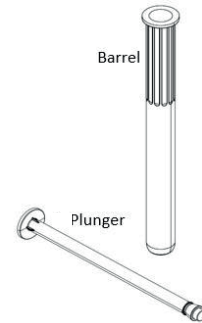


"Flip Open" to Open

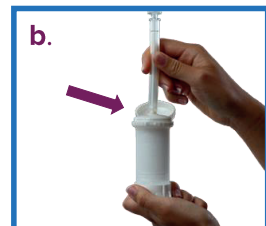

"Snap-On" to Load

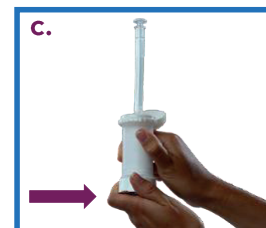

"Turn / CLICK" to Dispense

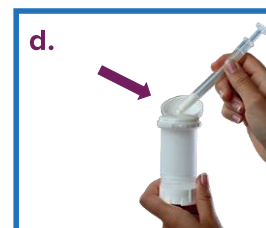

"Tilt-Off" to Remove

Perl Instructions012417

Improving Life Through Innovative Pharmacy Solutions

**DoseLogix.com | 877.870.8448 | info@DoseLogix.com**