

## Topi-CLICK® Filling Instructions

Read all information, specifications and instructions carefully before filling and dispensing Topi-CLICKs.



### STEP 1:

Fill the Topi-CLICK® dispenser with the desired amount of topical formulations using a Topi-CLICK® Filling Station, a spatula, EMP jar, or nozzle.



### STEP 2:

If needed while filling, stop periodically to tap the Topi-CLICK® on the palm of your hand, counter or Tap-n-CLICK™ to help settle the topical.



### STEP 3:

Put the applicator pad (cap with a hole) on the top of the Topi-CLICK® container by aligning the inside key tabs with the slots on the top of the container tube and snap on.



### STEP 4:

Prime the Topi-CLICK® by turning the base until topical is dispensed and wipe off topical.

Be sure to leave the base in the registered CLICK position by aligning the base column with the tube node: See Step 3 of the **Patient Use Instructions**.



Registered Position



### STEP 5:

Snap on the protective outer cap. (If the cap doesn't fit tight, it's an indication that the applicator pad in #3 is not "snapped" down all the way.)

*Note: Be sure to align ports when using the 3Port™ cap and applicator.*



### STEP 6:

Apply center of label to the side of the Topi-CLICK® container which is opposite the "REFILL" and graduation marks.

For more information on dispensing amounts per CLICK see the **Product Specification's** at DoseLogix.com. See **Patient User Instructions** for information on how to apply topical medications using Topi-CLICK. **Verify formulas and weight per volume as required for compounded prescription pharmaceuticals.**

Topi-CLICK® by DoseLogix is the most accurate topical dosing dispenser among those compared in an independent study titled *Topical Metered Dosing Dispenser Performance Evaluation* by Analytical Research Laboratories.

Request study at [http://DoseLogix.com/accuracy\\_study](http://DoseLogix.com/accuracy_study).

Topi-CLICKFillingInstructions011617

Improving Life Through Innovative Pharmacy Solutions

**DoseLogix.com | 877.870.8448 | info@DoseLogix.com**

## The Topi-Filler™ Filling Station

### For Fast and Easy Filling of Topical Formulations

The patented Topi-Filler™ Filling Station is designed to make filling Topi-CLICKs and other devices easy and more efficient.

- Most EMP jar sizes can be quickly interchanged.
- Optimizes the closed EMP system; less batch transfers and minimal storage costs.
- Works with right or left handed operators via switchable operating handle.
- Balanced actuation arm for easy operation.
- Made of durable stainless and powder coated steel for years of operation.
- Easy to clean and wipe down.
- Contrast background to help assure quality filling in see-through containers.
- Stabilizer plate for use when filling adapter type dispensers from EMP jars.
- Includes NEW nozzle specific for filling Topi-CLICK.



**Topi-Filler™ Filling Station**  
SKU#: TFS1000  
Description: Filling Station

## Topi-CLICK® Filling Nozzles

For the pharmacies using the Topi-Filler™ Filling Station, these inexpensive polypropylene nozzles are reusable and aid in increased workflow solution for the pharmacy.

### Benefits

- Inexpensive & re-usable
- Work with EMP filling jars
- Easy flow for thicker formulations
- Reduce the overlapping air while filling, reducing air bubbles



**Small Filling Nozzles**  
SKU#: TT1009  
Description: Small Filling Nozzle  
for filling the Topi-CLICK 35  
Quantity: 5/Pack



**Large Filling Nozzles**  
SKU#: TT1409  
Description: Large Filling Nozzle  
for filling the Topi-CLICK 140  
Quantity: 5/Pack

Visit [DoseLogix.com](http://DoseLogix.com) for more accessories to make filling topical formulations more efficient.

Topi-CLICK® by DoseLogix is the most accurate topical dosing dispenser among those compared in an independent study titled *Topical Metered Dosing Dispenser Performance Evaluation* by Analytical Research Laboratories.

Request study at [http://DoseLogix.com/accuracy\\_study](http://DoseLogix.com/accuracy_study).

Topi-CLICKFillingInstructions011617

Improving Life Through Innovative Pharmacy Solutions