


## Topi-CLICK® Patient Use Instructions



### STEP 1:

Lift off the cap of the Topi-CLICK® container by placing your thumb under the tab on the protective outer cap and pressing upward.

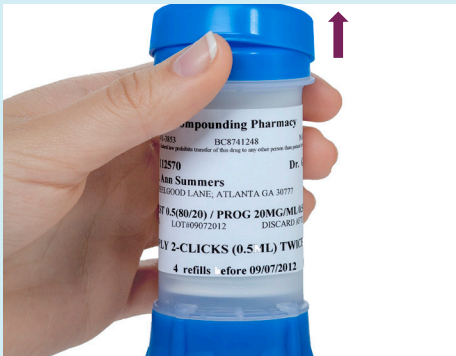

### STEP 2:

Turn the base to dispense the number of CLICKS specified in the instructions on your prescription label.



### STEP 3:


Watch the visual indicators when rotating base. Be sure to leave the base in the registered CLICK position by aligning the base column with the tube node after each use.



### STEP 4:

Apply topical medication by rubbing the area of skin as directed by your pharmacist using the **built-in applicator**. **Continue rubbing until topical is completely absorbed into the skin.**

**IMPORTANT:** The built-in applicator lessens the risk of cross-contamination and ensures the full dose is applied to the intended area.

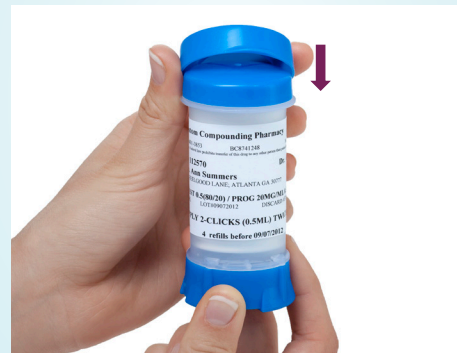

### STEP 5:

Replace cap after each dose to prevent evaporation. Be sure cap snaps on tightly. **Note** Be sure to align ports when using the Topi-CLICK® 35 3Port™ cap and applicator.



### STEP 6:

The bottom of the plunger is the indicating line; when it reaches the embossed line below the word "Refill" approximately 32 CLICKs are left. Approximately 12 CLICKs remain at the line above the word "Refill".

*Follow instructions on provider's label before using the Topi-CLICK®.*

Topi-CLICK® 35 is the most accurate topical dosing dispenser among those compared in an independent study titled *Topical Metered Dosing Dispenser Performance Evaluation* by Analytical Research Laboratories. Request study at [http://doselogix.com/accuracy\\_study/](http://doselogix.com/accuracy_study/).